PushBackVac

Congratulations!

You have purchased a product that will make your chairside dental care more efficient and easier. This device is made of polycarbonate and intended solely for dental use.

USE – DO NOT twist the suction into place holding the c-shaped end. This will place excessive torque on the suction and break it. Grip the device on the tube portion and twist to place. You can moisten the tube end to make this easier. Gently place the suction in the patient's mouth by retracting the cheek and depressing the tongue slightly.

CARE – Sterilize before first use. The PushBackVac dental suction should be cleaned thoroughly after each use preferably with a bottle brush. It can be sterilized with dental sterilizing solutions or by autoclaving. Autoclaving can make this device brittle over time. **DO NOT** sterilize using high heat. This product will melt.

END OF LIFE – This product is plastic and can be recycled. Sterilize before placing in this waste stream.

Please go to our website at PushBackVac.com for more information.

 Warning. This product may contain BisPhenol A (BPA) known to the State of California to cause cancer and birth defects or other reproductive harm. For more information go to P65warnings.ca.gov

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Special Dental Products, LLC
3250 Far Reach Drive
Baldwinsville, NY 13027
USA
315-853-7447

Contents: 2 suctions, two patient left